


Our Cord Blood Collection Systems at a glance:

THE BRIDGE TO GOOD HEALTHCARE




CBCS Premium Set A

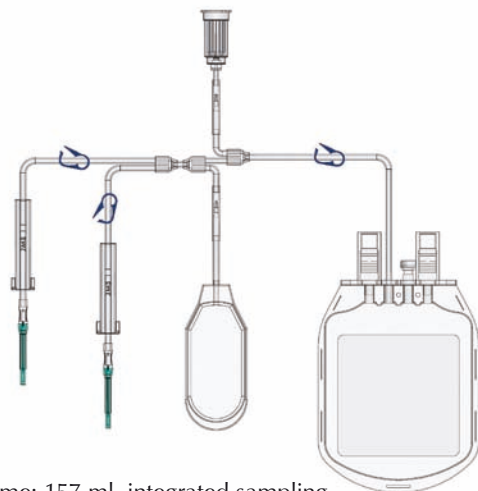


Volume: 157 ml, integrated sampling port, flushing bag and two needles.
Order Code: S811-1010.

CBCS Premium Set B

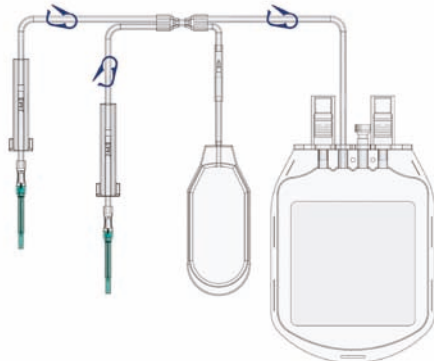

Volume: 157 ml, with flushing bag and two needles. Order Code: S811-1009.

CBCS Standard Set A

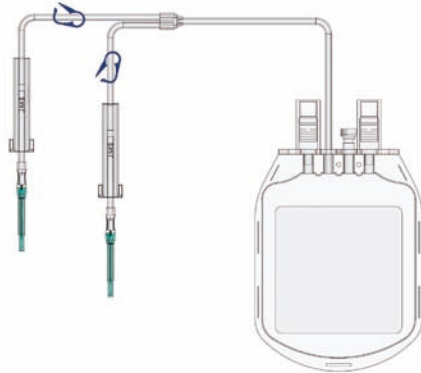

Volume: 157 ml, with two needles.
Order Code: S811-1000.

Safe Double Packaging



Double packaging in a transparent pack and an additional aluminum laminated pack to ensure maximal sterility.



Stem cell collection – safe and easy

The efficient and safe Cord Blood Collection System

For more information, please contact us:



Bionic Medizintechnik GmbH – a JMS corporation
Max-Planck-Str. 21 · D-61381 Friedrichsdorf, Germany
Telefon: +49(0)6172-75760 · Fax: +49(0)6172-757610
E-mail: info@bionic-jms.com · www.bionic-jms.com

Rights for technical modifications reserved
JMS-CBBE/01 2014-10


JMS Cord Blood Collection System – for a safer future

Optimal collection features for easy use



Optimal collection features allow a safe and effective procedure for cord blood collection:

- ▶ Close the clamp near the cannula. Remove the cap and insert the needle into the umbilical cord.
- ▶ Allow the blood collection by placing the collection bag lower than the injection site and opening the clamp.
- ▶ Mix blood and anticoagulant gently at several intervals.
- ▶ Close the clamp after the collection is finished.
- ▶ Residual blood in the tubing may be flushed by breaking the snap tip to open the optional flushing bag.
- ▶ Seal the tube and discard the tubing, flushing bag and needles.

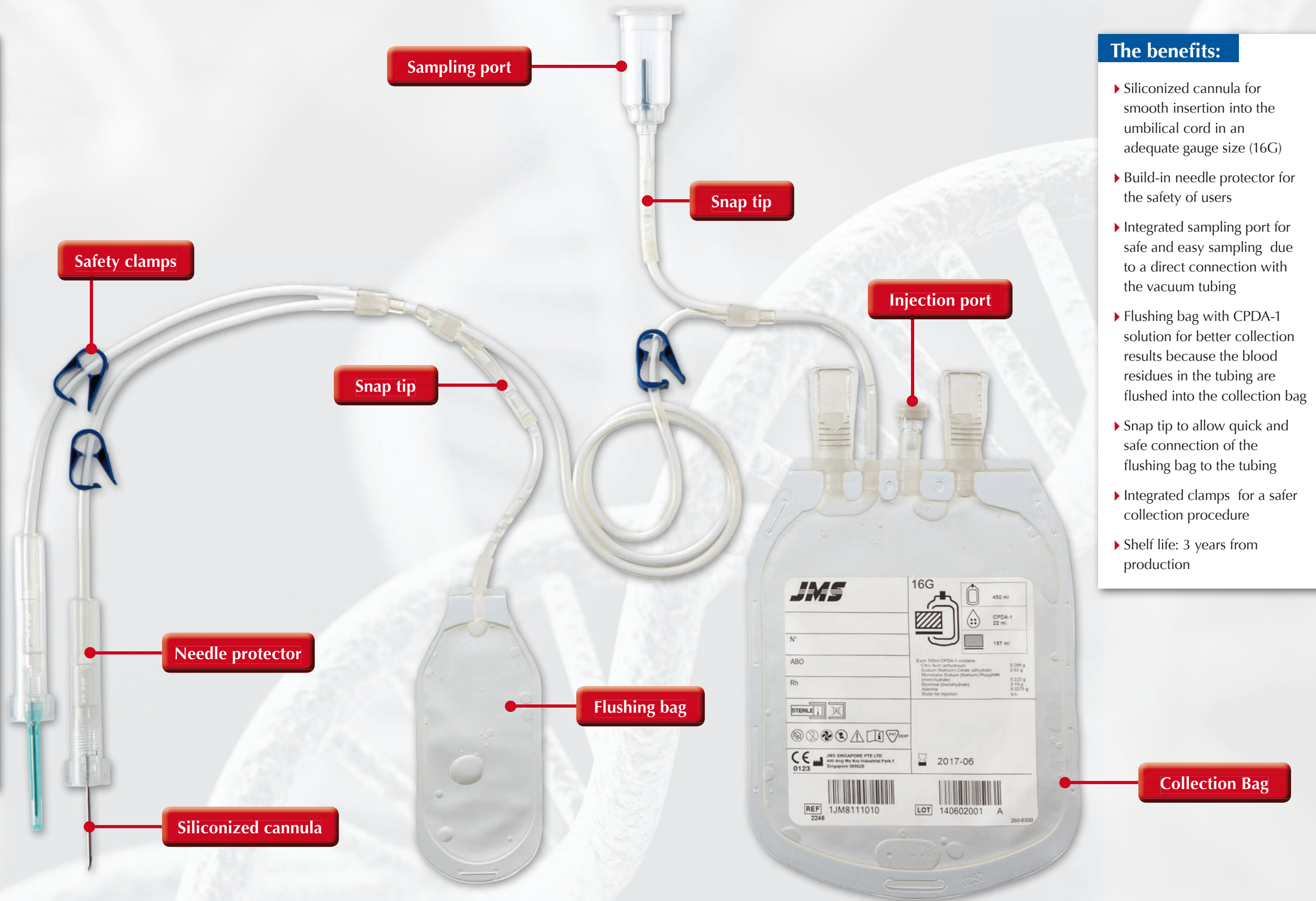

The benefits:

- ▶ Siliconized cannula for smooth insertion into the umbilical cord in an adequate gauge size (16G)
- ▶ Build-in needle protector for the safety of users
- ▶ Integrated sampling port for safe and easy sampling due to a direct connection with the vacuum tubing
- ▶ Flushing bag with CPDA-1 solution for better collection results because the blood residues in the tubing are flushed into the collection bag
- ▶ Snap tip to allow quick and safe connection of the flushing bag to the tubing
- ▶ Integrated clamps for a safer collection procedure
- ▶ Shelf life: 3 years from production